

Apex® EZTest®


Activation by Hand

1



Break media ampoule

2

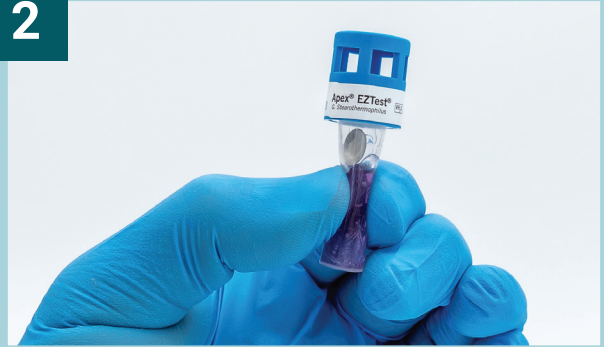


Crush glass until disc falls into media

3



Ensure disc is completely submerged in media

4

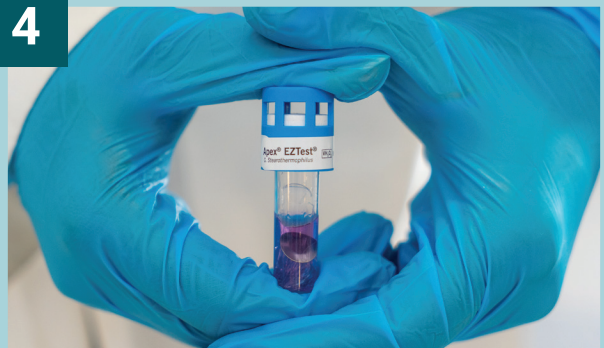

Depress cap until **click** is heard and felt

5



Incubate 7 days at 55 - 60°C

6




Media changes to yellow if spores survive, indicating a positive result

Apex® EZTest®


Activation by Crusher

1

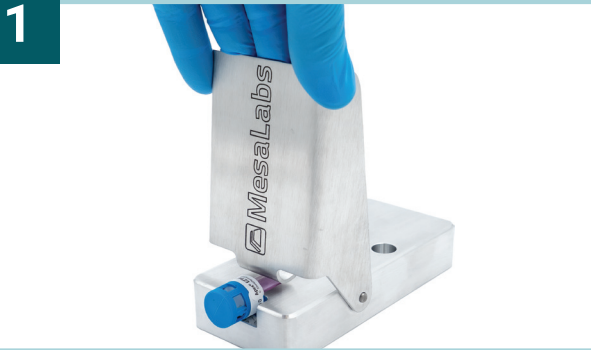

Break media ampoule

2

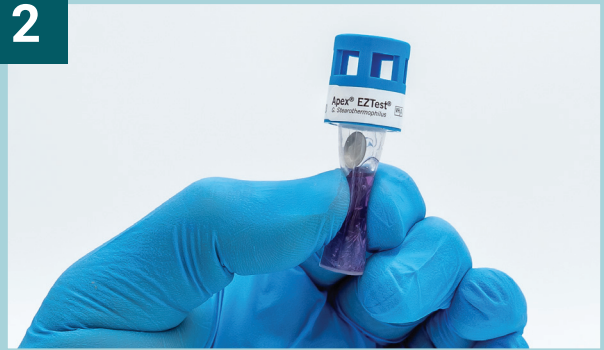

Crush glass until disc falls into media

3



Ensure disc is completely submerged in media

4



Depress cap until **click** is heard and felt

5



Incubate 7 days at 55 - 60°C

6



Media changes to yellow if spores survive, indicating a positive result